

canadianstage

The Sennheiser Mobile Connect App Assisted Listening on Your Smartphone



At home: Download the App




Search “Sennheiser Mobile Connect” in your app store to download.

Bring your own headphones to the theatre, or purchase earbuds from the box office for \$3




At the theatre: Connect to the Assisted Listening WiFi network at the theatre

Go to “Settings” App on your smartphone and choose the Assisted Listening local WiFi network.

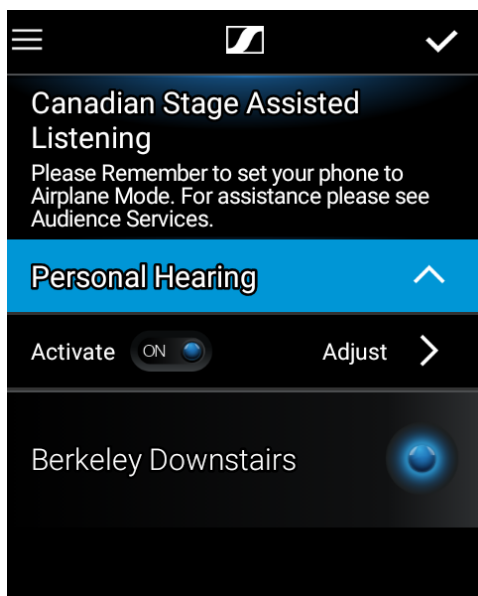


At the theatre: Start the App and adjust sound


After starting the app, select the Berkeley Downstairs channel. Use the volume buttons on your phone to adjust volume.

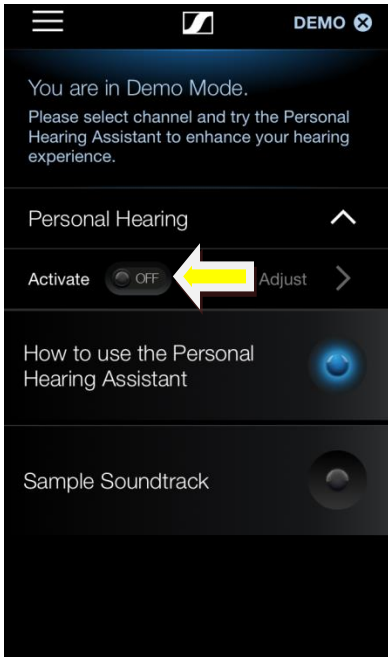

Click on Activate - the Personal Hearing Bar will become blue.

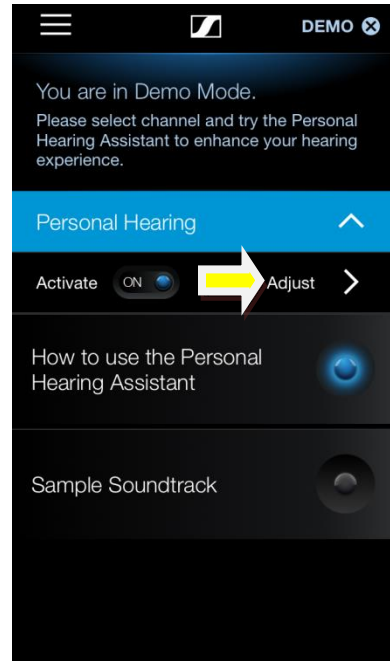

Click on Adjust to get to the sound adjusting screen.

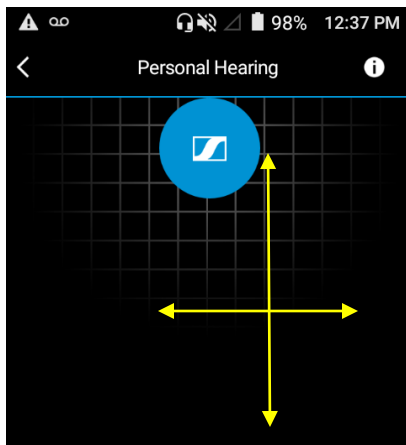

Dragging the circle up increases the volume, down decreases volume; to the left emphasizes bass, to the right emphasizes treble. Ideal position is top centre of the screen.